CE signs explanation:.....	2
GB: English - Product information – Master file for content	4
DE: German - Produktinformation.....	5
FR: French - Information concernant le produit.....	7
ES: Spanish - Información sobre el producto.....	9
IT: Italien - Informazioni sul prodotto.....	10
SLO: Slovenian - Navodilo za uporabo	12
PL: Polish - Informacja o produkcie.....	14
SK: Slovak - Informácie o výrobku.....	16
RO: Romanian - Instrucțiuni de utilizare	17
HU: Hungarian - Termékinformáció	19
CZ: Czech - Příbalová informace.....	21
LV: Latvian - Lietošanas instrukcija	22
LT: Lithuanian - Naudojimo Instrukcija.....	24
EST: Estonian - Kasutusjuhend	26
NL: Dutch - Productinformatie	27
RU: Russian – Информация о продукте	29
TR: Turkish - Ürün bilgileri	31
PT: Portuguese - Informação do produto	33
GR: Grec - Πληροφορίες προϊόντος.....	35
BG: Bulgarian – ПРОДУКТОВА ИНФОРМАЦИЯ	37
RS: Serbian - Uputstvo za upotrebu	38
HR: Croatian - Uputa za upotrebu	40
ARB: Arabian	45
FAR: Farsi	47

CE signs explanation:

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GB: English - Product information – Master file for content

Dianatal® obstetric gel - For an easier birth

For use by qualified obstetric professionals (midwives and physicians) only.

Please read this information before using Dianatal® obstetric gel.

Application form and contents:

Dianatal® obstetric gel Set: sterile single-use tubes containing 5 ml obstetric gel:

Properties, galenic form and ingredients

Dianatal® obstetric gel is packed in single-use tubes. Dianatal® is non-allergenic, bioadhesive, electroconductive, slightly acidic, free of preservatives and latex and does not cause irritation to mucous membranes and or eyes.

Dianatal® obstetric gel does not contain any pharmaceutically active ingredients.

Dianatal® Obstetric Gel is an odourless, almost colourless gel packed in sterile single-use packaging. The gel is highly viscous, with a slightly acidic pH of 5.5 – 6.5 and an electrical conductivity of $\geq 5 \text{ mS} \cdot \text{cm}^{-1}$.

Ingredients:

One single-use tube of 5 ml Dianatal® obstetric gel contains following ingredients:

Aqua purificata, Propylenglycol, Hydroxyethylcellulose, Sodium chloride, Carbomer, Sodium hydroxide

Indication and uses of Dianatal® obstetric gel

Dianatal® obstetric gel has been developed specifically to facilitate vaginal delivery. Dianatal® is bioadhesive and has a high water binding capacity. Dianatal® is used during vaginal delivery to make birth easier for both mother and child and to reduce perineal trauma. Dianatal® forms a low-friction bioadhesive film on the vaginal birth canal, reducing the friction forces between the child and the vaginal birth canal to facilitate vaginal delivery.

Dianatal® obstetric gel is sterile, non-allergenic, electroconductive, slightly acidic and does not cause irritation to mucosal membranes or eyes.

Dianatal® obstetric gel can be used in first-time mothers (randomized control trial (RCT) results) as well as in multiparas and in women with a history of cesarean section (clinical experience, investigator trial results).

Dianatal® obstetric gel eases and shortens both the first and second stages of labour and offers protection to mother and child; in particular protecting the vagina, the pelvic floor and the perineum.

Dianatal® obstetric gel is used specifically:

- to facilitate vaginal delivery and to protect both mother and child by reducing the opposing friction force in first-time mothers (RCT results)
- to facilitate vaginal delivery and to protect both mother and child by reducing the opposing friction force in multiparas, water deliveries and women with history of cesarean (clinical experience, investigator trial results)
- to prevent or treat prolonged or obstructed delivery (clinical experience, investigator trial results)
- to facilitate vaginal operative procedures (vacuum/forceps) (clinical experience)
- to protect the perineum and the vagina during vaginal delivery (RCT results)

Dosage and application

Dianatal® obstetric gel is applied after vaginal examination during delivery after onset of regular contractions, beginning with the vaginal examination around 3–4 cm dilatation of the cervix (active phase of labor). Approx. 3–5 ml birth gel is applied ahead of the presenting part with the tube or manually. An additional application of gel should be made 15–30 minutes after rupture of the amniotic sac.

Dianatal® obstetric gel shortens both the first and second stages of labour. It should therefore be used throughout the entire delivery. The dosage is, on average, between 15 ml and 25 ml Dianatal® obstetric gel per delivery. Experience has shown that the maximum amount required is 30 ml Dianatal® obstetric gel.

The application should only be performed by qualified obstetric professionals.

Method of use:

For local application in the vaginal birth canal only.

Precautions:

Content is sterile as long as the blister pack has not been opened. You should not use Dianatal® obstetric gel if the packaging is damaged, if the packaging has been opened or if the expiry date has been exceeded. Once the sterile blister pack has been opened the tubes must

be handled in accordance with sterile guidelines to maintain sterility. After delivery of the baby's head, the face area (eyes, mouth and nose area) should be wiped with a dry cloth. If appropriate, superficial cleaning of the mouth/nose area should also be carried out. It is advisable to use a dry towel to deliver the baby to prevent the baby from slipping out of the hands.

Dianatal® obstetric gel should not be used in combination with other gels as this may impair the electrical conductivity of Dianatal® obstetric gel.

Biocompatibility and use during pregnancy:

The biocompatibility of Dianatal® obstetric gel has been tested according to international guidelines. There is no evidence for any harmful effect on the mother or the baby.

Adverse effects

No adverse effects have been reported to date.

Interaction

No interaction events have been reported to date.

Contraindications:

Dianatal® obstetric gel should not be used if:

- there is the possibility of an allergic reaction to any of the ingredients
- vaginal delivery is contra-indicated

Storage and shelf-life:

Dianatal® obstetric gel should be stored at room temperature.

The expiry date is printed on the outer wrapping and packaging. Dianatal® obstetric gel should not be used after the expiry date.

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Please refer to the instructions for use Must not be reused

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Dianatal® is a registered Trademark of Tentan AG, Switzerland